

PSL classic

Quick guide to controls

Rotate eyepiece bodies to adjust PD

LED fixation targets

Click and hold the trigger to turn the lamp illumination on

2 x Double click and hold the trigger to turn lamp illumination and LED fixation targets on

Slit Wheel
 1x1 0.15 0.5 0.8 1.6 12

Filter Wheel - Red free, Blue, Neutral density (0.8) and Clear

Rotate eyepieces to focus the oculars

Magnification change lever
 16x 10x

Rotate wheel to adjust brightness

This LED comes on when charging

The battery lasts for more than 50 minutes

This LED flashes when the battery is almost empty

The battery can be charged by the docking station or device